

1 PRODUCT DESCRIPTION AND INTENDED PURPOSE

Timertag Go is a free label template that gives patients and families plain-language guidance about a medical consumable in place, such as a peripheral IV catheter, an indwelling urinary catheter, or a dressing. Timertag Go labels must be professionally printed from the template as 20 mm × 40 mm rounded-corner labels supplied on rolls. Each label carries a printed QR code. Scanning it with a smartphone camera opens a medical consumable-specific guidance page in the phone's web browser, built on language-agnostic graphics and a simple checklist of signs to report to nursing staff.

No app, no account, no login, and no patient data. Timertag Go labels are static: they have no timing or monitoring function. For dwell-time tracking, see Timertag Pro.

2 CLASSIFICATION NOTICE

Timertag Go labels are informational and educational aids only. They do not monitor the medical consumable or the patient, do not alert staff, do not process patient data, and do not replace clinical observation or judgment.

3 INTENDED USERS

For ordering by the deploying organization and application by clinical staff. For scanning by patients, families, and caregivers using any internet-enabled smartphone with a camera.

4 WHAT YOU NEED

- The current Timertag Go label template for the relevant medical consumable type, downloaded from www.timertag.com.
- A dedicated label printing company that can supply 20 mm × 40 mm rounded-corner rectangular labels on rolls.
- An internet connection at the point of care for the phone opening the guidance page.


5 ORDERING AND PRINTING

Download the current template for the relevant medical consumable type from www.timertag.com and supply it to your label printer unaltered. Labels must be produced as 20 mm × 40 mm rounded-corner rectangles on rolls. Do not resize, crop, or redraw the artwork: distortion of the QR code prevents scanning. Commercial printers apply their own cutting tolerances and color profiles, so review and approve the printer's proof before the production run. Test-scan labels from each delivered batch before clinical use. Reorder from the current template rather than reusing stored files, so guidance reflects the latest version.

6 APPLICATION

Apply the label to a clean, dry, visible surface near the medical consumable: the edge of the dressing for a peripheral IV, or the exterior of the drainage bag for a urinary catheter. Do not place the label over the insertion site, the inspection window of a dressing, or printed graduations. Replace the label if it becomes soiled, damaged, or unreadable.

WORKFLOW OVERVIEW

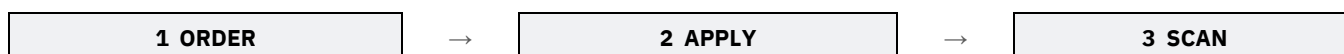

7 PATIENT USE

Open the phone's camera, point it at the QR code, and tap the link that appears. The guidance page explains key warning signs to report to nursing staff immediately: looseness, leakage, swelling, redness, pain, or fever.

8 WARNINGS AND PRECAUTIONS

! READ BEFORE USE

- **Guidance only.** Timertag Go does not observe the medical consumable or the patient and does not notify staff. Clinical observation schedules remain unchanged.
- **Single-patient use.** Do not move a label between patients.
- Do not apply the label to skin or over broken skin.
- Specify non-metallic, non-foil label stock when ordering.

9 MATERIALS AND MRI NOTE

As supplied by Timertag, a Go label consists of printed artwork only and contains no electronic, metallic, or magnetic components. Printed on standard non-metallic label stock, the label presents no known hazard in the MRI environment.

10 DATA, PRIVACY, AND TERMS OF USE

No accounts, logins, or patient identifiers. As with any webpage, standard technical information (such as IP address and browser type) is processed to deliver the page. Downloading and printing Timertag Go is subject to the Timertag Download and Usage Agreement (Version 2026.2), reproduced in Annex A of this document and published at www.timertag.com. Free for use in patient care. Not for resale.

11 DISPOSAL

Dispose of used labels as general waste, or per facility clinical waste procedures if contaminated with blood or body fluids.

Senver Group Pty Ltd (ABN 90 622 761 380), trading as Timertag
53/61–65 Glencoe Street, Sutherland NSW 2232, Australia
www.timertag.com

TIMERTAG®, Timertag Pro, and Timertag Go are trademarks of Senver Holdings Pty Ltd, used under license by Senver Group Pty Ltd. © 2026 Senver Group Pty Ltd. All rights reserved. IFU-TTG-001 Rev A, issued 2026-07. English (EN).